

# Make your practice more profitable by getting reimbursement for what you already do.



If you send out drug tests to the lab, you can now bill for on-site urine testing. Utilize the Multidrug Onsite Test Cup one-step drug cup.

After urine specimen is collected, wait five minutes and read the test strips for results. Then place urine sample in shipping container and send to our lab for confirmation of strip results. Reimbursement per test cup to your office is \$136.00 on average. CPT code 80101 QW Modifier. Example reimbursement for one patient tested would be \$17.11 per drug class (figure based on reimbursement of 8 possible drug classes). Drug classes tested for are the following.

Drug Class	Abbreviations	Sensitivity
Amphetamine	AMP	1000 ng/ml
Barbiturates	BAR	300 ng/ml
Benzodiazepines	BZD	300 ng/ml
Cocaine/Benzoyllecgonine	COC/BEG	300 ng/ml
Marijuana	THC	50 ng/ml
Methadone	MAD	300 ng/ml
Methamphetamine	MET	1000 ng/ml
Opiates/Morphine	OPI/MOR	2000 ng/ml
Oxycodone	OXY	100 ng/ml
Phencyclidine	PCP	25 ng/ml
Tricyclic Antidepressant	TCA	1000 ng/ml

Drug abuse remains a growing social and economical concern. The above stated drugs are among the most frequently abused drugs, according to the U.S. Substance Abuse and Mental Health Services Administration (SAMHSA) and the U.S. Department of Health and Human Services.

*Note: The test provides only preliminary data which should be confirmed by other methods such as gas chromatography/mass spectrometry (GC/MS). Clinical considerations and professional judgment should be applied to any drug of abuse test result, particularly when preliminary positive results are indicated.*

## Principle of the Test

The Multidrug Onsite Test Cup is a competitive binding immunoassay in which drug and drug metabolites in a urine sample compete with immobilized drug conjugate for limited labeled antibody binding sites. By utilizing antibodies that are specific to different drug classes, the test permits independent, simultaneous detection of any of the drug combinations from a single sample. The approximate run time is 5 minutes.

In the assay procedure, urine mixes with labeled antibody-dye conjugate and migrates along a porous membrane. When the concentration of a given drug is below the detection limit of the test, unbound antibody-dye conjugate binds to antigen conjugate immobilized on the membrane, producing a rose-pink color band in the appropriate Test Zone for that drug. Conversely, when the drug level is at or above the detection limit, free drug competes with the immobilized antigen conjugate on the membrane by binding to antibody-dye conjugate, forming an antigen-antibody complex, preventing the development of a rose-pink color band.

Test is reimbursed by most commercial health plans and Medicare. State sponsored health plans are reimbursed based on the individual contracts with the providers. CLIA waiver for office is required and assistance will be provided.

Let us do the billing and add no additional work to your staff. Billing is offered for the drug testing at 3% from our specialty billing company, which is sometimes less expensive than can be done in house. Call the number to the left of our logo now for information on Multidrug Onsite Test Cup.

**Is your office complying with the most recent policy statement from the Drug Enforcement Agency?**